


### Step 1: Prep the application site

- Trimming excess hair on application site
- Wipe with BioPlus+ & allow to dry (15 seconds)
- Optional: wipe application site with alcohol wipe for patients with oily skin.




### Step 2: Apply hydrocolloid oval

- Rest CathGrip on the prepped application site.
- Remove the release liners, pressing the hydrocolloid to the skin as you go
- Hold for 10-15 seconds to secure the seal

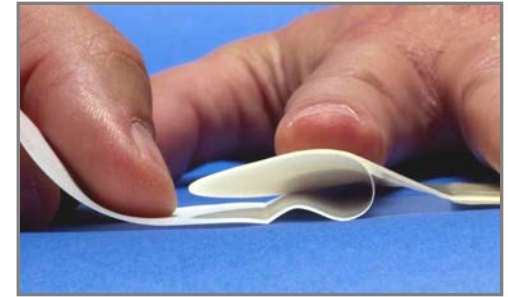

### Step 3: Secure tube with soft, no-slip grips

- Rest the tube across the no-slip grips.
- If using Grip & Seal, wrap the hydrocolloid wings around the base of the tube at the application site.
- Slide the grip through the corresponding hold and tighten until the tube is secure.
- If using Hold Me, rest the second hydrocolloid over the top of the tube and remove the release liners. Press and hold the hydrocolloid sandwich to secure.

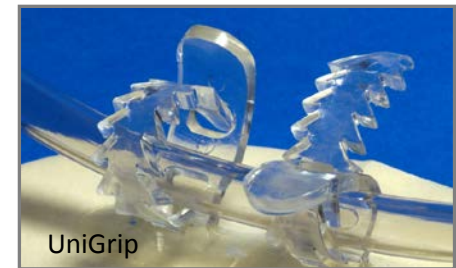

Grip & Seal



Hold Me