

FOR BEST RESULTS:

Clean application site with an alcohol wipe and/or *non-moisturizing* soap prior to application.

Contents:

- 1 Alcohol Wipe
- 1 BioPlus+™ Barrier Wipe
- 2 Hydrocolloid Ovals

For use on intact skin.

If redness or irritation occurs, stop use and consult a healthcare practitioner.



CathGrip®
HoldMe™



See reverse for complete application instructions.



Manufactured by BioDerm®, inc.

12320 73rd Court N, Largo, FL, USA

Orders: (800) 814-3134 Support: (800) 864-6755 Web: www.BioDermInc.com

©2017 BioDerm, Inc. All rights reserved. CathGrip® and BioDerm® are registered trademarks of BioDerm, Inc. HoldMe™ and BioPlus+™ are trademarks of BioDerm, Inc. CathGrip U.S. patent pending.

LT 90368 D


Applying HoldMe™




CathGrip®
HoldMe™




STEP 1: Clean entire application site with alcohol wipe and allow to dry.

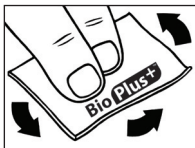

STEP 2: Wipe the entire application site with BioPlus+™. Allow 15+ seconds to dry.

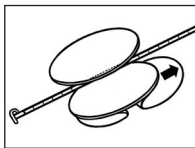

STEP 3: Place the first oval and remove the release papers. Press to the skin.

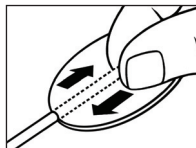

STEP 4: Rest the tube on the oval. Press second oval on top and remove release papers. Press to secure.

ATTENTION: Following application, leave a one inch space around the CathGrip® device when using any cleansing products, creams or lotions. Contact with the hydrocolloid will result in early dislodgement of the securement device.