

FOR BEST RESULTS:


Clean application site with an alcohol wipe and/or **non-moisturizing** soap prior to application.

Contents:

- 1 Alcohol Wipe
- 1 BioPlus+™ Barrier Wipe
- 1 Oval Universal Securement Device
- 1 Slotted Hydrocolloid Oval

For use on intact skin.

If redness or irritation occurs, stop use and consult a healthcare practitioner.

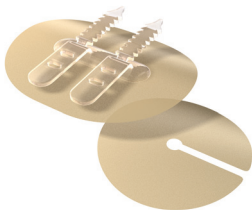

See reverse for complete application instructions.



Manufactured by BioDerm®, inc.

12320 73rd Court N, Largo, FL

Orders: (800) 814-3134 Support: (800) 864-6755 Web: www.BioDermInc.com

©2017 BioDerm, Inc. All rights reserved. CathGrip® and BioDerm® are registered trademarks of BioDerm, Inc. PEG Protect™, UniGrip™ and BioPlus+™ are trademarks of BioDerm, Inc. CathGrip U.S. patent pending.

LT 90370 B


Applying PEG Protect™




CathGrip®
PEGProtect™




STEP 1: Clean entire application site with alcohol wipe and allow to dry.

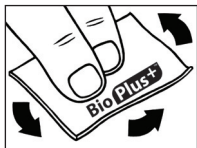

STEP 2: Wipe the PEG site and UniGrip™ application site with BioPlus+™. Allow 15+ seconds to dry.

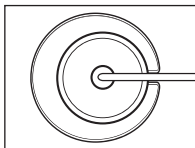

STEP 3: Place the slotted oval underneath the PEG bumper. Remove the release papers. Press to the skin.

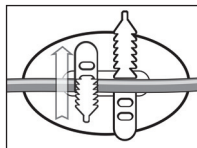

STEP 4: Place UniGrip 4"-6" from the insertion site. Remove release papers and press to the skin. Secure tubes with no-slip grips.

ATTENTION: Following application, leave a one inch space around the PEG Protect device when using any cleansing products, creams or lotions. Contact with the hydrocolloid will result in early dislodgement of the securement device.