

Designed to temporarily stop the flow of urine through the urethra. MRI Compatible.

PRECAUTIONS:

Do not use continuously. Do not apply too tightly as this may reduce blood flow or cause damage to the penis. **DISCONTINUE USE** if you experience pain or skin irritation at the clamp site. Discontinue use if blood is noticed in urine. Consult your healthcare professional in all instances.

KindKlamp™



SKU# 20003



Manufactured by BioDerm®, inc.

12320 73rd Court N, Largo, FL

Orders: 800-373-7006 Support: 800-864-6755 Web: www.BioDermInc.com

© 2017 BioDerm, Inc. All rights reserved. BioDerm® is a registered trademark of BioDerm, Inc. Kind Klamp™ and Men's Liberty™ are trademarks of BioDerm, Inc. Kind Klamp U.S. patent pending.

LT 94068 E

Directions for Use:

If using with the Men's Liberty™, begin by washing your penis with Ivory or Gold Bar Dial soap and allow to dry. If not, please skip this step.

1. Dry your anatomy with a clean towel and immediately place the Kind Klamp™ close to the base of the penis, pressing it down to the first catch point. **Make sure the shaft of your penis is between the soft pads.** If the urine flow stops, you are ready to proceed to the next step. If it does not stop, continue tightening, one click at a time, until the flow stops.
2. After leakage stops, take a dry tissue and gently squeeze the glans to express any remaining urine.
3. Complete steps one through four on the Men's Liberty instructions as written.
4. After application is complete, remove the Kind Klamp and clean as per the instructions to the right.



Cleaning Your Kind Klamp:

1. The Kind Klamp may be washed in the sink with warm water and dish or hand soap.
2. Rinse thoroughly with cool water.
3. Gently squeeze the foam to remove excess water.
4. Allow to air dry at room temperature.

Do not use bleach, disinfectants or hair dryers. Do not place in the dishwasher or clothes washer.