

FOR BEST RESULTS:


Clean application site with an alcohol wipe and/or *non-moisturizing* soap prior to application.

Contents:

- 1 Alcohol Wipe*
- 1 BioPlus+™ Barrier Wipe
- 1 Winged Hydrocolloid Securement Device

For use on intact skin.

If redness or irritation occurs, stop use and consult a healthcare practitioner.

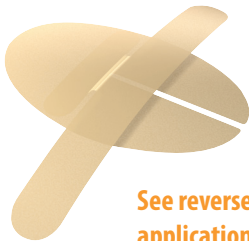

See reverse for complete application instructions.



Manufactured by BioDerm®, inc.

12320 73rd Court N, Largo, FL, 33773, USA

Orders: (800) 814-3134 Support: (800) 864-6755

Web: www.BioDermInc.com

**Not included
in part numbers
ending with NA*

Applying WingSeal™




CathGrip®
WingSeal™




STEP 1: Clean entire application site with alcohol wipe and allow to dry.



STEP 2: Wipe the entire application site with BioPlus+™. Allow 15+ seconds to dry.

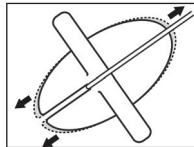

STEP 3: Place the WingSeal™ at the insertion site and remove the release papers. Press to skin.

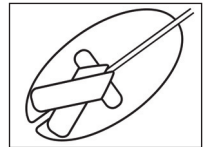

STEP 4: Wrap wings around the tube in an "X" shape. Pinch tube wrap to secure.

ATTENTION: Following application, leave a one inch space around the device when using any cleansing products, creams or lotions. Contact with the hydrocolloid will result in early dislodgement of the securement device.